Stool Preparation Kit

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Stool Preparation Kit

Stool Preparation Kit for the extraction of pancreatic elastase from stool samples of human origin for use with the IVD product Pancreatic Elastase ELISA

| | | |
|------------|------------------------------------|-----------|
| REF | ACS-001 | 45 pcs. |
| IVD | In-vitro-diagnostic medical device | CE |

Seramun Diagnostica GmbH • Spreehagener Str. 1 • 15754 Heidesee • Germany •
T +49 33767 791-10 • info@seramun.com • www.seramun.com

IVD In-vitro diagnostic medical device

 Country of manufacture and date of manufacture

 Keep away from sunlight

 Consult instructions for use

 Sufficient for n tests

UDI Unique device identifier

REF Article number

 Humidity limitation

 Temperature limit

 Biohazard

 Manufacturer

SN Serial number

LOT Batch code

 Do not reuse

 Use-by date

 Attention

Intended Use

Stool Preparation Kit is an IVD accessory for the extraction of pancreatic elastase from stool samples of human origin for the IVD product Pancreatic Elastase ELISA by a laboratory professional user.

The product must not be used with specimen materials other than stool samples of human origin, for the extraction of analytes other than pancreatic elastase, in the near patient setting and by lay persons.

Principle of the Test

The used principle of the test in which the Stool Preparation Kit is used is explained in more detail in the instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

Test Components (Delivery Scope)

| | | For 45 stool samples |
|---|--------------------------------------|---|
| 1 | Stool Preparation Kit TUBE | 45 systems for sample preparation with 1.5 mL extraction buffer, colorless, blue/yellow cap |
| 2 | Certificate of Analysis | 1 piece |
| 3 | Instructions for Use | 1 piece |

Additional Materials and Aids Required for the Test Procedure

Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS)

Important Information

These components are for *in-vitro* diagnostic use only. Follow the instructions carefully.

The components may be used by laboratory professionals only.

Do not use reagents from damaged packages or bottles. Imprinted expiry dates must be observed. Do not mix components with reagents from other manufacturers.

Mixing of components of different lots is not allowed.

All serious incidents occurring in relation with Stool Preparation Kit must be reported to the manufacturer and the competent authority of the EU member state in which user and/or patient are located.

Information on Assay Procedure

All reagents should be stored at 2...8 °C. Bring all components to room temperature before use.

Reagents that appear contaminated should not be used.

Follow the working scheme and time schedules of the protocol.

Safety Instructions

Reagents must not be swallowed. Contact with skin or mucous membranes should be avoided. Some reagents may contain biocides as preservative. Handle all components and patient samples as if potentially hazardous and infectious.

Additional information may be taken from the Material Safety Data Sheet.

Product contains the following hazardous component/-s:

| Component | Number | Full text |
|-----------|--------|-----------|
| TUBE | - | - |

Limitations of the Procedure

The Stool Preparation Kit is to be used only in combination with the IVD product Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

At temperatures higher than 40 °C, samples should be transported refrigerated or frozen. Aqueous stool samples may give falsely low results due to the dilution effect. In the case of watery stool, obtain another sample with a more solid consistency. If this is not possible, 15 µL of the watery stool can be pipetted directly into the extraction buffer by unscrewing the blue middle part together with the yellow top part (sample dilution 1:101, 15 µL stool + 1.5 mL extraction buffer). Since there is a risk of false pathological (falsely low) values due to the dilution of the stool, the consistency must be noted and in case of a pathological finding, the test must be repeated with a solid stool sample.

Sample Treatment

Sample Collection

Collect stool sample in suitable sampling container.

Sample Shelf Life and Storage

Stool samples can be stored at different temperatures for the following time periods.

| <u>native human stool sample</u> | | <u>extracted sample supernatant</u> | |
|----------------------------------|-----------------|-------------------------------------|-----------------|
| <u>storage</u> | <u>duration</u> | <u>storage</u> | <u>duration</u> |
| 2...8 °C | 7 days | 2...8 °C | 3 days |
| 20...22 °C | 5 days | -18...-27 °C | 1 month |
| 40 °C | 2 days | | |

Stool samples and extracted sample supernatant can be frozen and thawed up to two times. Therefore, it is recommended to aliquot samples to avoid frequently freeze and thaw cycles.

Sample Preparation

Using the Stool Preparation Kit:

1. Marking of the extraction tubes with patient data.
2. Unscrewing the yellow cap and pulling out the integrated rod.
3. Dipping the rod into the stool sample up to the last notch with drill-like movements.
4. Place the stool-loaded stick through the blue middle part into the tube and close tightly.
5. Mixing stool sample with the extraction buffer in the tube for at least 2 min (e.g. vortexing). The sample dilution here is 1:101 (15 mg stool + 1.5 mL extraction buffer).
6. Incubation of the suspension for at least 15 min at room temperature or overnight at 2 – 8 °C.
7. Mixing stool sample with the extraction buffer in the tube for at least 2 min (e.g. vortexing) again.
8. Sedimenting the stool particles for 15 minutes.
9. Unscrewing the blue middle part together with the yellow top part without stirring up sediment and discard.
10. Collection of the supernatant for further analysis.

Note: If salt crystals have precipitated in the extraction buffer, please dissolve them before use by gently shaking and heating in a water bath at 30 - 35 °C. For later analyses, the collected supernatants can be stored at approx. -18 °C for 4 weeks. It is important that the supernatants are properly separated from the sediment during storage.

Note: Further sample treatment is performed according to the instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

Reagent Treatment

Reagent Shelf Life and Storage

The Stool Preparation Kit with sealed reagent bottles can be stored at 2...8 °C until the printed expiration date.

Reagent Preparation

Bring Stool Preparation Kit to room temperature before use.

Assay Procedure

Note: The assay procedure is performed according to the instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

Evaluation of Results

Note: The evaluation of results is performed according to the instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

Interpretation of Results

Note: The interpretation of results is performed according to the instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS).

Performance Characteristics

Accuracy of measurement (weighing of stool vs Stool Preparation Kit)

To determine the mean percentage deviation of the measured pancreatic elastase concentration between stool sample weighing and the use of the Stool Preparation Kit, n=51 samples were comparatively tested in the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS). The mean percentage deviation in pancreatic elastase concentration between stool sample weighing and Stool Preparation Kit was 7.8 %.

Precision (precision of extraction)

To determine the precision of extraction (Intra-Assay-, Inter-Assay- and Lot-to-Lot-coefficient of variation) of the Stool Preparation Kit and further measurement with the Pancreatic Elastase ELISA, 8 samples were extracted and determined on 5 days of two operators in duplicates within 3 batches of the Stool Preparation Kit. As an example, the data generated for the Pancreatic Elastase ELISA (G09040) are shown here.

| Sample | Intra-Assay-coefficient of variation | | Inter-Assay-coefficient of variation | | Lot-to-Lot-coefficient of variation | |
|--------|--------------------------------------|--------|--------------------------------------|---------------------------------|-------------------------------------|---------------------------------|
| | \bar{x} µg elastase / g stool | CV (%) | \bar{x} µg elastase / g stool | \bar{x} µg elastase / g stool | CV (%) | \bar{x} µg elastase / g stool |
| 1 | 109.3 | 4.3 | 109.3 | 16.1 | 104.3 | 16.0 |
| 2 | 229.4 | 3.0 | 229.4 | 4.1 | 255.5 | 15.6 |
| 3 | 341.8 | 1.6 | 341.8 | 6.4 | 326.0 | 7.2 |
| 4 | 324.5 | 4.6 | 324.5 | 9.8 | 331.7 | 9.8 |
| 5 | 385.8 | 1.4 | 385.8 | 7.3 | 366.4 | 9.7 |
| 6 | 456.5 | 1.3 | 456.5 | 4.7 | 433.9 | 7.4 |
| 7 | 458.3 | 2.0 | 458.3 | 5.8 | 461.3 | 4.2 |

Note: Further performance characteristics have been determined in combination with the IVD product Pancreatic Elastase ELISA. Please refer to instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS) for more information on the performance characteristics.

Change History

| Version | Section | Modifications |
|-------------------|-----------------|---|
| 2022-09_v01_DE_EN | Entire document | Updating of the intended use Conversion of subsections Insertion of safety instructions Insertion of performance characteristics |

References

Please refer to instructions for use of the Pancreatic Elastase ELISA (G09040, G09038, G09038USA and G09038RUS) for more information on the references.