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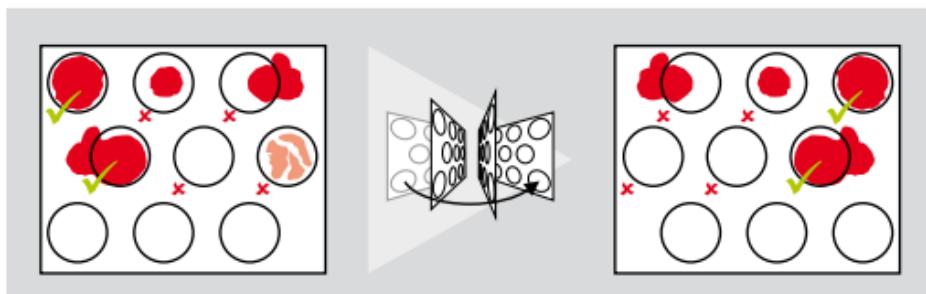
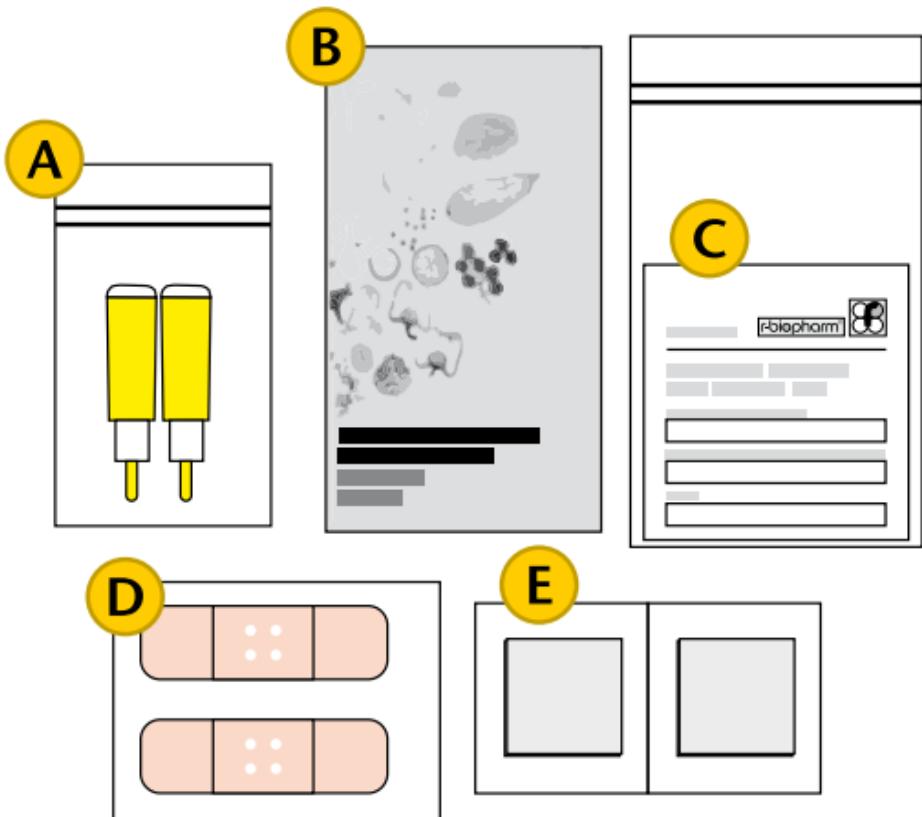
# RIDASCREEN® Foodscreen Blood Collection Kit

**REF** Art. Nr. A8025, A8025-IAH, A8025-IMU, A8025-PIM

The RIDASCREEN® Foodscreen Blood Collection Kit is a tool for the collection and transport of a human capillary blood sample from the fingertip. The kit is intended for self-testing.

## 1. General information

The RIDASCREEN® Foodscreen Blood Collection Kit allows an easy collection of capillary blood samples. The samples can then be sent to a laboratory for further analysis and testing for food intolerances.

## 2. Contents

- A** Two sterile lancets\*   
- B** Instructions for use
- C** RIDASCREEN® Foodscreen Blood Collection Card
- D** Two bandages\*
- E** Two alcohol swabs\*

Illustrated instructions for the test procedure

\* Manufacturer: see packaging

### 3. Procedure (also see illustrated instructions for the test procedure)

1. Please write your first and last name on the card **C** along with the blood collection date. Fill out the forms provided by the laboratory, if applicable. Please do not use a liquid ink roller pen because the ink can “bleed” through the paper.
2. Warm your hands under warm water for several minutes. Dry your hands
3. Allow your arm to hang and shake your fingers to stimulate blood flow. Choose a fingertip (preferably your ring or middle finger). Gently massage the entire length of the finger while holding your hand lower than your heart.
4. Twist off the yellow protective cap from the lancet **A** and pull it out. The second lancet is a spare.
5. Clean the finger with the alcohol swab **E** and allow it to dry completely.
6. Position the lancet **A** on your fingertip (ring or middle finger) slightly off center and press down firmly against the puncture site to activate the device.

Nederlands	Dansk	ру́сский	Italiano	Français	Português	Español	English	Deutsch	Norsk
Čeština	ελληνικά	Suomi	Polski	Român	Türk	български	Svenska		

7. Massage the entire length of the finger to obtain a large drop of blood.
8. At least 4 circles must be filled completely. Touch the tip of the blood drop to the first circle **C**. Do not touch the card with your finger! It may take several drops of blood to fill the circle all the way to the edge. Repeat this procedure for the other circles. If there is not enough blood, return to step 2 and repeat the process with another finger. Use the spare lancet.
9. Check that at least 4 circles are correctly filled (see inside of envelope). Also check the back of the card to ensure the circles are soaked through.
10. Apply a bandage **D** to the puncture site.
11. Allow the card **C** to air-dry overnight. Once it is completely dry, fold the card and place it in the aluminum bag with the desiccant
12. Put the aluminum bag with the card **C** and any completed laboratory forms in the envelope and mail it to the laboratory.

## 4. Warnings and precautions for users

- The RIDASCREEN® Foodscreen Blood Collection Kit should be stored dry in the original packaging between 9°C and 30°C.
- The laboratory must receive your sample within two weeks from the date of collection.
- Do not use this kit for blood withdrawal if you are taking blood thinning medications or if you have a blood clotting disorder unless you consult with your physician or therapist.
- The lancets provided are sterile, single-use lancets. Once the finger is punctured, the blade is automatically retracted into the sheath for safety.
- The lancets have a puncture depth of 2.0 mm. Caution is advised for use on young children: The distance between the finger surface and the bone may not be sufficient and using the lancet could result in injury. Consult your physician or therapist.
- Use in children under the age of one year is not recommended.

Nederlands	Dansk	ру́сский	Italiano	Français	Português	Españól	English	Deutsch	Norsk
Čeština	ελληνικά	Suomi	Polski	Român	Türk	български	Svenska		

- Do not use open or damaged kits to collect blood.
- The RIDASCREEN® Foodscreen Blood Collection Kit may not be used after the expiration date on the package.
- All the components in the kit are for single use only.
- Anyone helping you take the blood sample should wear disposable gloves (not contained in the kit).
- Do not take blood samples from a finger with visible injuries, the earlobe, or the little finger.
- Keep away from children. Minors should not use the kit without parental consent.
- If you do not understand the instructions, please consult with your physician or pharmacist.

- R-Biopharm AG expressly rejects any liability for the use or sale of the RIDASCREEN® Foodscreen Blood Collection Kit or its components if used for anything other than the intended use specified in the instructions.
  - The RIDASCREEN® Foodscreen Blood Collection Kit is an accessory for the RIDASCREEN® Spec. IgG Food screen and RIDACHIP® FoodGuide and was validated with those products. R-Biopharm AG assumes no liability for use with non-validated test systems.

## 5. Explanation of symbols

Consult instructions for use

Lot number

Expiry

Store at

Article number

Number of tests

Date of manufacture

Manufacturer

Do not reuse

CE label

Sterilized with radiation

Keep dry

Use only if packaging  
is intact

## 6. Version history

Version number	Chapter and designation	Nederlands	Dansk	ру́сский	Italiano	Français	Português	Español	English	Deutsch	Norsk
2019-03-08	Initial version	Čeština	Ελληνικά	Suomi	Polski	Român	Türk	Български	Svenska		
2019-11-01	Revision										
	New layout; new article added; intended use modified; general information, symbols for kit components added; warning and precautions for users, procedure, symbol explanation expanded; illustrated instructions for test procedure added.										

# RIDASCREEN® Foodscreen Blood Collection Kit

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