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# RIDASCREEN® Foodscreen Blood Collection Kit

**REF** Art. Nr. A8025, A8025-IMU, A8025-PIM

For *in vitro* diagnostic use. The RIDASCREEN® Foodscreen Blood Collection Kit/Card is used for collecting and transporting a human capillary blood sample taken from the fingertip. For professional use only.

## 1. General information

The RIDASCREEN® Foodscreen Blood Collection Kit allows for easy collection of capillary blood samples. The samples can then be sent to a laboratory for further analysis and testing for food intolerances.

## 2. Reagents provided

- A** Two sterile lancets\*
- B** Instructions for use
- C** RIDASCREEN® Foodscreen Blood Collection Card
- D** Two bandages\*\*
- E** Two alcohol swabs\*

Illustrated instructions for the test procedure

\* Manufacturer: see packaging

\*\* Manufacturer: BSN medical GmbH

### 3. Procedure (also see illustrated instructions for the test procedure)

1. Please write your first and last name on the card **C** along with the blood collection date. Fill out the forms provided by the laboratory, if applicable. Please do not use a liquid ink roller pen because the ink can “bleed” through the paper.
2. Warm your hands under warm water for several minutes. Dry your hands.
3. Allow your arm to hang and shake your fingers to stimulate blood flow. Choose a fingertip (preferably your ring or middle finger). Gently massage the entire length of the finger while holding your hand lower than your heart.
4. Twist off the yellow protective cap from the lancet **A** and pull it out. The second lancet is a spare.
5. Clean the finger with the alcohol swab **E** and allow it to dry completely.
6. Position the lancet **A** on your fingertip (ring or middle finger) slightly off center and press down firmly against the puncture site to activate the device.

Deutsch	Magyar
English	Lietuvių k.
Español	Latviešu valoda
Português	Norsk
Italiano	Nederlands
Français	Polski
Български	Română
Čeština	Русский
Dansk	Slovenščina
Ελληνικά	Slovenščina
Eesti	Svenska
Suomi	Türkçe

- Massage the entire length of the finger to obtain a large drop of blood.
- Touch the tip of the blood drop to the first circle **C**. Do not touch the card with your finger! It may take several drops of blood to fill the circles all the way to the edge. Repeat this procedure for the other circles. If there is not enough blood, return to step 2 and repeat the process with another finger. Use the spare lancet. You need at least the following number of circles for the various test parameters:

Testsystem	
RIDA <sup>®</sup> CHIP FoodGuide	RIDASCREEN <sup>®</sup> Spec. IgG Foodscreen
2 circles	<ul style="list-style-type: none"> <li>• ≤ 100 allergens: 4 circles</li> <li>• ≤ 200 allergens: 6 circles</li> <li>• ≤ 300 allergens: 8 circles</li> </ul>

- Check to make sure the circles are filled correctly (see inside of envelope). Also check the back of the card to ensure the circles are soaked through.
- Apply a bandage to the puncture site **D**.
- Allow the card **C** to air-dry overnight. Once it is completely dry, fold the card and place it in the aluminum bag with the desiccant.
- Put the aluminum bag with the card **C** and any completed laboratory forms in the envelope and mail it to the laboratory.

#### 4. Warnings and precautions for users

- The RIDASCREEN® Foodscreen Blood Collection Kit should be stored dry in the original packaging between 9°C and 30°C.
- The laboratory must receive your sample within two weeks from the date of collection.
- Do not use this kit for blood withdrawal if you are taking blood thinning medications or if you have a blood clotting disorder unless you consult your physician or therapist.
- The lancets provided are sterile, single-use lancets. Once the finger is punctured, the blade is automatically retracted into the sheath for safety.
- The lancets have a puncture depth of 2.0 mm. Caution is advised for use on young children: The distance between the finger surface and the bone may not be sufficient and using the lancet could result in injury. Consult your physician or therapist.
- Use in children under the age of one year is not recommended.
- Do not use open or damaged kits to collect blood.
- The RIDASCREEN® Foodscreen Blood Collection Kit may not be used after the expiration date on the package.
- All the components in the kit are for single use only.
- Anyone helping you take the blood sample should wear disposable gloves (not contained in the kit).
- Do not take blood samples from a finger with visible injuries, the earlobe, or the little finger.
- Keep away from children. Minors should not use the kit without parental consent.
- If you do not understand the instructions, please consult your physician or pharmacist.
- Users are responsible for the proper disposal of all materials after use. For disposal, please adhere to national regulations.
- R-Biopharm AG expressly rejects any liability for the use or sale of the RIDASCREEN® Foodscreen Blood Collection Kit or its components if used for anything other than the intended use specified in the instructions.
- The RIDASCREEN® Foodscreen Blood Collection Kit is an accessory for the RIDASCREEN® Spec. IgG Foodscreen and RIDACHIP® FoodGuide test systems and was validated with them. R-Biopharm AG assumes no liability for use with non-validated test systems.
- For users in the European Union: Report all serious adverse events associated with the product to R-Biopharm AG and the appropriate national authorities.

Deutsch	Magyar
English	Lietuvių k.
Español	Latviešu valoda
Português	Norsk
Italiano	Nederlands
Français	Polski
Български	Română
Čeština	Русский
Dansk	Slovenčina
Ελληνικά	Slovenščina
Eesti	Svenska
Suomi	Türkçe

## 5. Explanation of symbols

 Observe operating manual	 Manufacturer
 Lot number	 Do not reuse
 Use before	 CE mark
 Storage temperature	 Sterilized with radiation
 Item number	 Keep dry
 Number of tests	 Use only if packaging is intact
 Date of manufacture	 Distributor
 Packaging system with single sterile barrier and protective outer packaging	 Medical device
 Avoid light exposure	

## 6. Version history

Version number	Chapter and designation
2022-03-09	IVDR Version

Suomi	Eesti	Ελληνικά	Dansk	Čeština	Български	Français	Italiano	Português	Español	English	Deutsch
Türkçe	Svenska	Slovenščina	Slovenčina	Русский	Română	Polski	Nederlands	Norsk	Latviešu valoda	Lietuvių k.	Magyar

# RIDASCREEN® Foodscreen Blood Collection Kit

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