



### PRODUCT INFORMATION

**Product Name:**

PROCEEDx Influenza A and Adenovirus Positive Sample

**Catalogue Number:**

VP-13-04

**Product Description:**

This product contains inactivated influenza A and adenovirus in storage buffer.

### SPECIFICATIONS

**Strains:**

*Influenza A/Texas/1/77 (H3N2)*, ICTV 00.046.0.01.001

*Human adenovirus 2, strain Adenoid 6*, ICTV 00.001.0.01.010.00.102.006

**Storage:**

Store this preparation at 2 °C to 8 °C.

**Shelf Life:**

72 weeks from manufacturing date.

**Volume:**

1 mL

### USE

**Instructions For Use:**

Use PROCEEDx product as per sample handling instructions specified by device or instrument manufacturer.

**Intended Use:**

The product is to be used **for immunoassay** instrument (device) verification/validation, manufacturing QMS, educational and R&D purposes.

**NOT FOR USE IN DIAGNOSTIC PROCEDURES. FOR RESEARCH USE ONLY.**

### ASSITANCE

If you have any questions regarding the production, testing or use of this antigen, please send them together with any relevant data to Microbix Technical Services, at [customer.service@microbix.com](mailto:customer.service@microbix.com) or fax 905-361-8911. Your complete satisfaction with the performance of this product is important to us.

### HAZARDS




We are aware of no specific biological hazards associated with this product. The preparation has been inactivated and should contain no infectious material however it should be handled as if potentially infectious. Generally accepted good laboratory practices appropriate to biological reagents should be employed. The buffer solution contains 0.02% sodium azide (CAS No. 26628-22-8) as a preservative.

### SPECIAL HANDLING

No special handling required for this product.

# QUALITY CONTROL INFORMATION

Test Kit	Target	Result
OSOM Flu A & B Catalogue # 190	Influenza A & B	<b>Influenza A Positive</b>  <b>Influenza B Negative</b>
Quidel Flu A & B Catalogue # 20183	Influenza A & B	<b>Influenza A Positive</b>  <b>Influenza B Negative</b>
Xpect Flu A & B Catalogue # R24600	Influenza A & B	<b>Influenza A Positive</b>  <b>Influenza B Negative</b>
Combi-Strip (Rota and Adeno) Catalogue # C-1004	Rotavirus & adenovirus	<b>Adenovirus Positive</b>  <b>Rotavirus Negative</b>
SAS Adeno Catalogue # 067020	Adenovirus	<b>Adenovirus Positive</b>
Xpect Flu A & B Catalogue # R24600	Influenza A & B	<b>Influenza A Positive</b>  <b>Influenza B Negative</b>
Xpect RSV Catalogue # R24601	Respiratory Syncytial Virus	<b>RSV Negative</b>
OSOM Flu A & B Catalogue # 190	Influenza A & B	<b>Influenza A Positive</b>  <b>Influenza B Negative</b>
Quidel Flu A & B Catalogue # 20183	Influenza A & B	<b>Influenza A Positive</b>  <b>Influenza B Negative</b>

REF	Catalog Number
LOT	Lot Number
	Temperature Limitation
	Expiration Date
	Biological Risk

