



PRODUCT INFORMATION

Product Name:

PROCEEDx Influenza A and Influenza B Positive Sample

Catalogue Number:

VP-13-02

Product Description:

This product contains inactivated influenza A virus and influenza B virus in storage buffer.

SPECIFICATIONS

Strains:

Influenza A/H1N1 Subtype

Influenza B/Hong Kong/5/72. ICTV 00.046.0.04.001

Storage:

Store this preparation at 2 °C to 8 °C.

Shelf Life:

72 weeks from manufacturing date.

Volume:

1 mL

USE

Instructions For Use:

Use PROCEEDx product as per sample handling instructions specified by device or instrument manufacturer.

Intended Use:

The product is to be used **for immunoassay and/or nucleic acid test** instrument (device) verification/validation, manufacturing QMS, educational and R&D purposes.

NOT FOR USE IN DIAGNOSTIC PROCEDURES. FOR RESEARCH USE ONLY.

ASSISTANCE

If you have any questions regarding the production, testing or use of this antigen, please send them together with any relevant data to Microbix Technical Services, at customer.service@microbix.com or fax 905-361-8911. Your complete satisfaction with the performance of this product is important to us.

HAZARDS

We are aware of no specific biological hazards associated with this product. The preparation has been inactivated and should contain no infectious material however it should be handled as if potentially infectious. Generally accepted good laboratory practices appropriate to biological reagents should be employed. The buffer solution contains 0.02% sodium azide (CAS No. 26628-22-8) as a preservative.

SPECIAL HANDLING

No special handling required for this product.

QUALITY CONTROL INFORMATION



Test Kit	Target	Result
Binax Now Influenza A & B Catalogue # 416-000	Influenza A & B	Influenza A Positive Influenza B Positive
Quick Vue Flu A/Flu B Catalogue # 20183	Influenza A & B	Influenza A Positive Influenza B Positive
Binax NOW RSV Catalogue # 430-022	Respiratory Syncytial Virus	Negative
SAS Adeno Catalogue # 067020	Adenovirus	Negative
SAS Rota Catalogue # 069020	Rotavirus	Negative

qPCR Test Kit	Analyte	Result
Xpert FLU/RSV Cepheid	Influenza A	Detected
Xpert FLU/RSV Cepheid	Influenza B	Detected
Xpert FLU/RSV Cepheid	Respiratory Syncytial Virus	Not Detected

REF	Catalog Number
LOT	Lot Number
	Temperature Limitation
	Expiration Date
	Biological Risk

