



PROCEEDTM FLOQ[®]

PRODUCT INFORMATION SHEET

Product Name: PROCEEDxTM FLOQ[®] SARS-CoV-2 Swab Positive Sample

Catalogue Number: VP-S-19-01

Storage: Store this preparation at 2 °C to 30 °C.

Description: This product contains human fibroblast cells and Severe Acute Respiratory Syndrome Coronavirus 2 (SARS-CoV-2) whole genome cDNA desiccated on a swab.

Intended Use: The product is to be used as a positive test sample in polymerase chain reaction (PCR) based tests for SARS-CoV-2 detection. The product is to be used for nucleic acid test instrument (device) verification/validation, manufacturing QMS, educational and R&D purposes.

NOT FOR USE IN DIAGNOSTIC PROCEDURES. FOR RESEARCH USE ONLY.

Handling: It is recommended that the product is tested according to kit Manufacturer's instructions for testing of clinical samples. collected by swab procedures. Generally, swab elution in 1-3 mL sample transport medium or H₂O is acceptable.

Expected Results: Testing of the product in RT-PCR-based assays for detection of SARS-CoV-2 should result in positive finding, including such assays where the presence of human DNA is required for the determination of a positive result.

Hazards: We are aware of no specific biological hazards associated with this product. The preparation has been inactivated and should contain no infectious material however it should be handled as if potentially infectious. Generally accepted good laboratory practices appropriate to biological reagents should be employed. The coating buffer solution contains 0.02% sodium azide (CAS No. 26628-22-8) as a preservative.

Strain/Cell line: NC_045512 genome/ MRC-5

Assistance: If you have any questions regarding the production, testing or use of this material, please send them together with any relevant data to Microbix Technical Services, at customer.service@microbix.com or fax 905-361-8911. Your complete satisfaction with the performance of this product is important to us.

Quality Control Information

Product Name: PROCEEDx™ FLOQ® SARS-CoV-2 Swab Positive Sample

Lot Number: 19S10001N

Product Code: VP-S-19-01

qPCR Test Kit	Target	Result
CDC 2019-Novel Coronavirus (2019-nCoV) based RT-PCR test for detection of SARS-Cov-2	SARS-CoV-2	Positive
	Human DNA	Positive
Xpert Flu/RSV	Influenza A	Negative
	Influenza B	Negative
	RSV	Negative

Inactivation: Product was inactivated.

Quality Control Summary: This lot meets specifications and is released for shipment.

Expiry date: 2021-04-30

Quality Assurance Signature:



Date: 06 July, 2020



*FLOQ® is a trademark of Copan Italia Spa.