



PRODUCT INFORMATION SHEET

Product Name: PROCEEDTMFLOQ[®] SARS-CoV-2 Ag Swabs Positive Samples

Catalogue Number: VP-S-19-02*

Storage: Store this preparation at 2 °C to 30 °C.

Description: This product consists of swabs containing desiccated SARS-CoV-2 nucleocapsid antigen on Copan FLOQSwabs[®].

Intended Use: The products are to be used as positive test swab samples in nucleoprotein-based immunoassay tests for SARS-CoV-2. They are to be used on immunoassay test instruments (devices) for verification/validation, manufacturing QMS, educational and R&D purposes.

***PRODUCT IN ADVANCED DEVELOPMENT
NOT FOR USE IN DIAGNOSTIC PROCEDURES. FOR RESEARCH USE ONLY**

Handling: It is recommended that the product is tested according to kit Manufacturer's instructions for testing of clinical swab samples.

Instructions For Use: When including PROCEEDTMFLOQ[®] SARS-CoV-2 Ag Swabs Positive Samples in a test run, the exact same procedure for unknown specimens collected on a swab must be used. Refer to the manufacturer supplied instructions for use provided with the SARS-CoV-2 test kit.

- Elute the swab by referring to the preferred technique described in the assay instructions for use (usually 0.25-0.5 mL).
- Use 100-200 µL from the eluted swab sample for the detection step.

Expected Results: Testing PROCEEDTMFLOQ[®] SARS-CoV-2 Ag Swabs Positive Samples using immunological assays for detection of SARS-CoV-2 nucleocapsid protein should result in a positive finding.

Hazards: We are aware of no specific biological hazards associated with this product. The preparation contains no infectious material. Generally accepted good laboratory practices appropriate to biological reagents should be employed. The coating buffer contains 0.02% sodium azide (CAS No. 26628-22-8) as a preservative.

Antigen: SARS-CoV-2 nucleocapsid protein

Assistance: If you have any questions regarding the production, testing or use of this material, please send them together with any relevant data to Microbix Technical Services, at customer.service@microbix.com or fax 905-361-8911. Your complete satisfaction with the performance of this product is important to us.

Quality Control Information

Product Name: PROCEEDx™FLOQ® SARS-CoV-2 Ag Swabs Positive Samples **Lot Number:**

Product Code: VP-S-19-02

Test Kit	Target	Result
Quidel Sofia 2 SARS	SARS-CoV-2	Positive

Inactivation: SARS-CoV-2 Ag related Products are non-infectious.

Expiry date:

Quality Assurance Signature:

Date:



Explanation of symbols used in Microbix product labelling:



Research use only



Caution, consult accompanying documents



Instructions for use



Use By



Temperature limitation



Manufacturer



Single use only



Contains sufficient for 'n' tests



Catalogue number



Lot number

FLOQ® is a trademark of Copan Italia Spa