



TECHNICAL INFORMATION SHEET

Product Name: ONBOARD™ Respiratory Ag swab kit

Catalogue Number: VP-SK-RESPAG-02

Storage: Store this preparation at 2 °C to 30 °C.

Description: This product contains twenty swabs in total, with five SARS-CoV-2 nucleocapsid protein positive swabs, five Influenza A positive swabs, five Influenza B positive swabs and five Respiratory negative swabs desiccated on Copan FLOQSwabs®.

Amount:

5x PROCEEDx™ FLOQ® SARS-CoV-2 Ag Swabs Positive Samples	(Cat# VP-S-19-02)
5x PROCEEDx™ FLOQ® Influenza A Positive Swab Positive Samples	(Cat# VP-S-13-01)
5x PROCEEDx™ FLOQ® Influenza B Swab Positive Samples	(Cat# VP-S-14-01)
5x PROCEEDx™ FLOQ® Respiratory Swab Negative Samples	(Cat# VP-S-99-01)

Intended Use: The products are to be used as positive test swab samples in antigen-based tests for SARS-CoV-2, and Influenza A&B (Flu A&B). They are to be used on antigen test instruments (devices) for verification/validation, manufacturing QMS, educational and R&D purposes.

Intended Use: The product is to be used as a positive and negative test swab sample in antigen detection-based tests for Flu A, Flu B and SARS-CoV-2. The product is to be used for immunological test instrument (device) verification/validation, manufacturing QMS, educational and R&D purposes.

NOT FOR USE IN DIAGNOSTIC PROCEDURES. FOR RESEARCH USE ONLY

Handling procedure: When including the PROCEEDx™ FLOQ® Swab Positive Samples and Swab Negative Sample in a test run, the exact same procedure for unknown specimens collected in on a swab must be used. Refer to the manufacturer supplied instructions for use provided with the SARS-CoV-2 test kit.

- Elute the REDx™ FLOQ® SARS-CoV-2 Ag Swab Positive Control by referring to the preferred technique and volumes described in the assay instructions for use (usually 0.2-0.7 mL).
- After elution, proceed with detection step by using the test volume specified in the assay instruction for use (usually 0.05-0.2 mL from the eluted swab volume).

Expected Results: Testing of the PROCEEDx™ FLOQ® SARS-CoV-2 Ag Swab Positive Sample in immunological assays for SARS-CoV-2 should result in a positive finding, testing of the PROCEEDx™ FLOQ® Respiratory Swab Negative Sample in immunological assays for SARS-CoV-2 should result in a negative finding.

Testing of the PROCEEDx™ FLOQ® Influenza A Swab Positive Sample in immunological assays for Influenza A should result in a positive finding.

Testing of the PROCEEDx™ FLOQ® Influenza B Swab Positive Sample in immunological assays for Influenza B should result in a positive finding.

Hazards: We are aware of no specific biological hazards associated with this product. The preparation has been inactivated and should contain no infectious material however it should be handled as if potentially infectious. Generally accepted good laboratory practices appropriate to biological and chemical reagents should be employed. The coating buffer 0.02% sodium azide (CAS No. 26628-22-8) as a preservative.

Strain/Antigen: SARS-CoV-2 nucleocapsid protein, Influenza A/H1N1, Influenza B/Hong Kong

Assistance: If you have any questions regarding the production, testing or use of this material, please send them together with any relevant data to Microbix Technical Services, at customer.service@microbix.com or fax 905-361-8911. Your complete

satisfaction with the performance of this product is important to us.

Quality Control Information

Quality Control Information

Product Name: PROCEEDx™ FLOQ® SARS-CoV-2 Ag Swab Positive Sample

Product Code: VP-S-19-02

Test Kit	Target	Result
Quidel Flu+SARS FIA	SARS-CoV-2	Positive
	Influenza A	Negative
	Influenza B	Negative

Product Name: PROCEEDx™ FLOQ® Influenza A Swab Positive Sample

Product Code: VP-S-13-01

Test Kit	Target	Result
Quidel Flu+SARS FIA	SARS-CoV-2	Negative
	Influenza A	Positive
	Influenza B	Negative

Product Name: PROCEEDx™ FLOQ® Influenza B Swab Positive Sample

Product Code: VP-S-14-01

Test Kit	Target	Result
Quidel Flu+SARS FIA	SARS-CoV-2	Negative
	Influenza A	Negative
	Influenza B	Positive

Product Name: PROCEEDx™ FLOQ® Respiratory Swab Negative Sample

Product Code: VP-S-99-01

Test Kit	Target	Result
Quidel Flu+SARS FIA	SARS-CoV-2	Negative
	Influenza A	Negative
	Influenza B	Negative

Inactivation: Product were inactivated.



Explanation of symbols used in Microbix product labelling



Research use only



Caution, consult accompanying documents



Instructions for use



Use By



Temperature limitation



Manufacturer



Single use only



Contains sufficient for 'n' tests



Catalogue number



Lot number

FLOQ® is a trademark of Copan Italia Spa