



PRODUCT INFORMATION

Product Name:

PROCEEDxTMFLOQ[®] SARS-CoV-2 Ag Swab Positive Sample

Catalogue Number:

VP-S-19-02

Product Description:

This product contains SARS-CoV-2 nucleocapsid protein desiccated on a swab.

SPECIFICATIONS

Antigen material:

SARS-CoV-2 nucleocapsid protein

Storage:

Store this preparation at 2 °C to 30 °C.

Shelf Life:

1 year from manufacturing date.

USE

Instructions For Use:

When including the PROCEEDxTMFLOQ[®] SARS-CoV-2 Ag Swab Positive Sample in a test run, the exact same procedure for unknown specimens collected in on a swab must be used. Refer to the manufacturer supplied instructions for use provided with the SARS-CoV-2 Antigen test kit.

Intended Use:

The product is to be used **for immunodiagnostic** instrument (device) verification/validation, manufacturing QMS, educational and R&D purposes.

NOT FOR USE IN DIAGNOSTIC PROCEDURES. FOR RESEARCH USE ONLY.

ASSITANCE

If you have any questions regarding the production, testing or use of this antigen, please send them together with any relevant data to Microbix Technical Services, at customer.service@microbix.com or fax 905-361-8911. Your complete satisfaction with the performance of this product is important to us.

HAZARDS

There are no specific hazards associated with this product. Generally accepted good laboratory practices appropriate to biological and chemical reagents should be employed. The coating buffer 0.02% sodium azide (CAS No. 26628-22-8) as a preservative. Please refer to the Safety Data Sheet (SDS).

HANDLING PROCEDURE



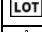


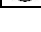
- Elute the PROCEEDxTMFLOQ[®] SARS-CoV-2 Ag Swab Positive Sample by referring to the preferred technique and volumes described in the assay instructions for use (usually 0.2-0.7 mL).
- After elution, proceed with detection step by using the test volume specified in the assay instruction for use (usually 50-200 µL from the eluted swab volume).

QUALITY CONTROL INFORMATION

Target	Result
SARS-CoV-2	Positive

EXPECTED RESULTS

Testing of the product in immunoassay-based tests for SARS-CoV-2 should result in a positive finding for SARS-CoV-2 nucleocapsid protein.

 RUO	Research Use Only
 REF	Catalog Number
 LOT	Lot Number
 Temperature Limitation	
 Expiration Date	
 Single Use Only	

